

2.0 STARTING AND CONNECTION

Unpacking

Take the cabinet out of the package carefully , so check any eventual damages caused by the transport. Position the cabinet on a flat surface, close to air connection and the electrical supply connection. Position the air control pedal on the ground.

Compressed air connection

Connect compressed air connection to the compressed air system.

Important: If it will be possible, connect the cabinet to a dry air compressor, in order reduce the humidity risk.

Filling of the containers

Unscrew the wheel closing of the relevant container. Full the container up to the indicated level (**no further**) . Remove any sand residues from the edge of the container , then screw the wheel.

Air pressure adjustment

Press the pedal to make the pressure visible on the manometer. Adjust the pressure regulator by turning it clockwise to increase the pressure or turning the regulator anti-clock to reduce the pressure (min. 0.5 max 6 bar) . Check the pressure on the manometer.

Important: to change the pressure , pull the regulator slightly upwards, then then turning it as you need. Press again the regulator to block it.

Apparatus switching

Press the main green switch (lighting) and the aspirator red switch (aspiration) . The switch indicator lights must be alight.

2.1 Use

Selection sand

Select the container by the lever inside the cabin . The stickers indicating the sand size, correspond to those placed on the containers.

Important: every time you select the sand , the container must be not under pressure.

Sanding.

Take the selected handpiece , then press on the pedal.

Cleaning.

Use the air syringe to clean the sanded article.

When work is complete.

The red switch turns out the aspirator . The green switch turns out the halogen lamp.